

Instructions for use:

PUROPTICS Eye Gel

PUROPTICS eye gel is a long-lasting, gentle and comfortable night time gel for dry and tired eyes providing overnight hydration therapy for relief from ocular discomfort caused by dryness.

PUROPTICS eye gel helps to protect and moistens eyes to remove the discomfort and irritation throughout the night.

CONTENTS:

PUROPTICS eye gel contains Carbomer 0.2% with the preservative Cetrimide, in buffered isotonic system.

CONTRAINDICATIONS:

Hypersensitivity to Carbomer or to any of the ingredients listed below.

Other Ingredients: Sorbitol Solution 70%, Diethylenetriaminepentakis (Methylphosphonic Acid) Solution, Hypromellose. May contain Sodium Hydroxide to adjust pH.

INTENDED USE:

PUROPTICS eye gel is a specially formulated long-lasting, gentle and comfortable night time gel for dry and tired eyes providing overnight hydration therapy for relief from ocular discomfort caused by dryness. **PUROPTICS** eye gel helps to protect and moistens eyes to remove the discomfort and irritation throughout the night.

DIRECTION FOR USE:

Place 1 or 2 drops in the eye(s) as needed.



- Get the **PUROPTICS** eye gel and a mirror.
- Wash your hands.
- Take the tube and peel off the tamper evident snap collar.
- Open the cap, hold the tube, pointing down, between your thumb and middle finger.
- Tilt your head back. Pull down your eyelid with a clean finger, until there is a 'pocket' between the eyelid and your eye. The drop will go in there

(picture 1).

- Bring the tube tip close to the eye. Use the mirror if it helps.
 - Don't touch your eye or eyelid, surrounding areas or other surface with the tube tip. It could contaminate the product left in the tube.
 - Gently press on the base of the tube to release one drop of **PUROPTICS** Eye gel at a time.
 - Don't squeeze the tube, it is designed so that just a gentle press on the bottom is needed (Picture 2).
 - If you use the gel in both eyes, repeat the steps for your other eye.
 - Put the cap back on firmly immediately after use. If a drop misses your eye, try again.
 - White particles may appear on eye lashes post administration of product if the product oozes out of the eyes. Wipe areas around eye with clean cloth soaked in lukewarm water after administration of **PUROPTICS** eye gel if needed.
- PRECAUTIONS:**
- Do not use if allergic to any of the ingredients.
 - The tube tip remains sterile till the tube is opened.
 - To avoid contamination or possible eye injury, do not touch tube tip to any surface and avoid contact with the eye.
 - Close the cap tightly immediately after use.
 - Once opened, discard any unused content after 1 month.
 - Do not use after expiry date indicated on the carton & tube.
 - Do not use if package is damaged or the tamper evident seal is broken.
 - Discard the product content if it becomes cloudy or brown
 - Store in a dry, dark & well-ventilated place at a temperature not exceeding 25°C.
 - Allow 30 minutes between the administration of other eye products.
 - Remove contact lenses before use.
 - Consult your doctor, pharmacist or optician before you use **PUROPTICS** eye gel if:
 - You are pregnant or breast feeding
 - You are suffering from any other eye disease or disorder

- You are using any other medicines in your eyes

SIDE EFFECTS:

Common: Eye irritation (Including burning and discomfort), eye pain, eye pruritus and / or visual disturbance.

Blurring of vision immediately after use. Do not operate machinery, use electric tools or drive a car until clear vision is restored.

In the event that you experience side effects not mentioned in this leaflet, or if there is prolonged irritation or severe visual disturbance or any change in your general health, consult your doctor immediately.

EXPECTED GROUP OF PATIENTS FOR OCULAR LUBRICATING & COMFORT SOLUTION:

Children above the age of 5 years, adults & geriatric population, who are affected by dry eye (such as soreness, burning, irritation or dryness) or dry eye syndrome.

Dry eye syndrome, also known as chronic dry eye is characterized by a reduced ability to form tears or a reduction in the quality of those tears.

WARNING: If you experience any problems with this product please discuss with your pharmacist, GP or eye care specialist. Do not soak or disinfect your lenses in **PUROPTICS** Eye gel

Distributed by:

BIOVANTIC
Biovantic Pharma Limited
Kemp House, 160 City Road,
London, EC1V 2NX, UK.
Email: info@biovanticpharma.co.uk

INSTRUCTION FOR DISPOSAL:

The device should be disposed as per prevailing regulations where the device is used.

HOW SUPPLIED:

PUROPTICS eye gel is supplied in 10g pack size in a lami tube.

Note : This leaflet is specific to **PUROPTICS** Eye gel

KEEP OUT OF SIGHT AND REACH OF CHILDREN

DO NOT SWALLOW

NOT FOR INJECTION

FOR EXTERNAL USE ONLY

	Temperature limitation
	Do not use if package is damaged
	Do not Re-use
	Do not Re-sterilize
	Attention, See Instructions for use
	Method of sterilization using aseptic processing techniques
	Keep away from sunlight
	Manufacturing date
	Lot number
	Should not be used after specification month & date
	Authorized representative in the European community
	Reference number
	UK Representative

CE
2803

REF Issue No: 01
Date of issue: